

Nima Hygienhätta (hygiene cap)



The cap has been developed to be used on top of existing covers to improve hygiene in public settings where more than one person sits on the same cushion.

The cap is also usefully applied to cushions used by a single user where the cushion's cover often becomes unhygienic due to incontinence or spills.

The cap can easily be wiped clean or disinfected after incontinence or spills. The smooth surface makes it harder for dirt and odour to take hold, and gives better resistance to bacterial growth than a standard fabric cover.

It is very easy to pull the cap over an existing cover on a cushion, which makes the process of removing, washing and refitting the cap considerably faster than the equivalent procedure with a cushion cover.

Material

The PU-coated polyester material is vapour-permeable (breathes) but protected against liquid. It is easy to wipe clean and keep odour-free. It is the same material normally used in healthcare hygiene mattresses.

User weight

Follow the instructions on the cushion used under the protection.

Fire tested for open flame and cigarette.
CE marked product.

Washing instructions

Machine wash at up to 90 degrees; can be dried in a drying cabinet or tumble-dryer.

User information/instructions for use/user guidance

The cap comes ready in its packaging. It is to be placed with the blue "rubber-like" side facing up.

Disposal

A cushion to be disposed of is sorted as combustible waste.