

Nima Kildyna (wedge cushion)

Wedge-shaped seat cushion for a forward-leaning sitting position



Intended use

Wedge-shaped seat cushion intended to give a forward-leaning sitting position. The high rear edge and low front edge give a slight forward tilt of the pelvis. The cushion is available with different inclinations depending on the user's needs. The foam material is light and soft. The cushion is also available in a version with soft foam at the centre rear for coccyx relief.

Use environment: chairs, armchairs and other seating where a forward-leaning sitting position is desired. **Users:** the user themselves, or care staff/relatives following a needs assessment by a licensed prescriber. **Delivery:** the cushion is delivered ready to use — check size and inclination when fitting. **Material:** cushion base of cold foam (density 55 kg/m³) in a wedge shape. Fire-retardant polyester cover.

Construction

Cold foam base (55 kg/m³) in a wedge shape — high rear edge, low front edge. Fire-retardant polyester cover. Article number states the dimensions: e.g. DKl40408020 is a 40×40 cm cushion with 8 cm at the highest point and 2 cm at the lowest.

How to use the cushion

1. Take the cushion out of its packaging.
2. Place the cushion according to the labelling — high edge to the rear, low edge to the front.
3. Check size and inclination — choose another model if the cushion does not suit the need.
4. The cushion is ready to use — no adjustment needed.

Reconditioning & care

Between users: wipe the cushion base with disinfectant or a 40 °C soap solution. Do not machine wash the cushion base.

Drying: dry the base flat in a drying cabinet or lying freely.

Cover: machine wash 60 °C, tumble-dry safe.

Disposal: a worn-out cushion is sorted as combustible waste.

Important to keep in mind

- The inclination must be chosen to suit the user's needs — the wrong inclination can give unstable sitting.
- Inspect the cushion and the skin regularly.

User weight	Max 150 kg (all sizes)
Material	Cold foam 55 kg/m ³ · Fire-retardant polyester cover
Design	Wedge shape — available with different inclinations
Sizes	Seat width/depth per article number (DKlIsbsd)
Weight	0.6–1.0 kg depending on size
Service life	Expected 5 years · Warranty 2 years

Accessories

- Standard polyester cover (*included*)
- Incontinence-protected cover
- Incontinence-protected cap with elastic

CE MD



Base:



Cover:



See page 2 for symbol explanations and full MDR info.

Technical documentation and MDR information

Supplementary regulatory information under Regulation (EU) 2017/745 on medical devices (MDR). Intended use, construction, user weight, accessories and care are stated on page 1.

Product identification and classification

Article number format: DKIsbsdshsh (where sb = seat width, sd = seat depth in cm, sh = rear height, sh = front height — e.g.

DKI40408020 = 40×40 cm, 8 cm rear / 2 cm front).

ISO code: 18 10 06 — Wedge-shaped seat cushion (ISO 9999).

Classification: Class I medical device under MDR (EU) 2017/745, rule 1.

Intended user: the user, following a needs assessment by a licensed prescriber.

UDI-DI: Each size has its own UDI-DI per GS1 (GTIN). Example: DKI40408020 → GTIN 7333376015709.

Barcode: EAN-13.

Sterile/reuse: Non-sterile; reusable between users after reconditioning as instructed on page 1.

Testing and standards

Fire tested per: ISO 7176-16 (resistance to ignition of upholstered wheelchair parts).

Symbols: Labelling per EN ISO 15223-1 (medical devices) and ISO 3758 (textile care).

Quality management: Manufactured under ISO 13485.

Declaration of conformity: The manufacturer has issued an EU declaration of conformity — contact Nimacare AB for the current version.

Restrictions

- Users with an **existing category 3 or 4 pressure ulcer** on the seating surface — require assessment by a licensed prescriber before use.
- User weight exceeding the permitted limit for the size in question (see page 1).
- Use of a damaged cushion.

Storage, inspection and environment

- Inspect the cushion and cover regularly — a damaged product must not be used.
- Avoid use near open flames or strong heat sources.
- Store cool and dry. Avoid prolonged direct sunlight.
- Do not use solvents for cleaning.

Modification

The cushion must **not be cut down**. If another size is needed — choose another size from the range. Modification or cutting voids the CE marking and releases the manufacturer from liability.

Residual risks

As with all seat cushions, a risk of pressure injury remains with prolonged uninterrupted loading or if skin checks are omitted.

Incorrect size or orientation can increase the risk of pressure injuries, unstable sitting and malpositioning. Users with reduced sensation or circulation require particular follow-up.

Reporting of serious incidents

Serious incidents related to the use of this product must be reported to Nimacare AB (info@nimacare.se) and to the competent authority of the Member State where the user and/or patient is established. In Sweden, incidents are reported to the **Swedish Medical Products Agency (Läkemedelsverket)** (www.lakemedelsverket.se).

Manufacturer and regulatory contact

Nimacare AB

Sigurdvägen 4, 18254 Djursholm, Sverige

SRN (EUDAMED): SE-MF-000050989

Tel: +46 (0)70 635 50 70 · E-mail: info@nimacare.se · Web:

www.nimacare.se

Person responsible for regulatory compliance (PRRC) under MDR Art. 15 — contact info@nimacare.se.

Symbol explanations

Medical device symbols per **EN ISO 15223-1**. Wash and care symbols per **ISO 3758**.



Medical device



Consult instructions for use



Wipe clean — base only, no machine wash



Dry flat — cushion base dried flat



CE marking — complies with MDR (EU) 2017/745



Manufacturer



Machine wash 60 °C — cover only



Tumble dry allowed — cover, normal temperature

Document information: This product sheet constitutes both product information and instructions for use in paper form per MDR (EU) 2017/745, Annex I, section 23. Latest revision: 2026-07-30. Document ID: NIMA-DKI-EN-IFU-Rev2026-07-30. If this version differs from a later version published at www.nimacare.se, the later version applies. The product is UDI-DI labelled — check that the UDI on the label matches the delivery.