

Nima Köksdyna (kitchen-chair cushion)

Kitchen-chair cushion in cold foam — rounded rear edge and fastening straps



Rounded rear edge — fits kitchen chairs

Intended use

Kitchen-chair cushion in cold foam intended for users who need a raised, comfortable sitting position to make standing up easier and to give added comfort when sitting. The material helps carry heat and moisture away from the seating surface. The cushion has a rounded rear edge that fits into kitchen chairs, and fastening straps that make it easy to attach the cushion to the chair's backrest.

Use environment: primarily kitchen chairs — can also be used in other seating. **Users:** the user themselves, or care staff/relatives following a needs assessment. **Delivery:** the cushion is delivered ready to use — no setting or adjustment required. **Material:** cushion base of cold foam (density 55 kg/m³). Fire-retardant polyester cover with two fastening straps.

Construction

Cold foam base (55 kg/m³) — firm, shape-stable foundation. Fire-retardant polyester cover with two fastening straps. Rounded rear edge adapted to kitchen chairs.

How to use the cushion

1. Take the cushion out of its packaging.
2. Place the cushion on the kitchen chair with the rounded edge to the rear, and fasten the straps around the backrest.
3. Check that the cushion's size fits the chair's seat — choose another size if needed.
4. The cushion is ready to use — no adjustment needed.

Reconditioning & care

Between users: wipe the cushion base with disinfectant or a 40 °C soap solution. Do not machine wash the cushion base.

Drying: dry the base flat in a drying cabinet or lying freely.

Cover: machine wash 60 °C, tumble-dry safe.

Disposal: a worn-out cushion is sorted as combustible waste.

Important to keep in mind

- The kitchen-chair cushion is **not a pressure relieving cushion** — choose another cushion where there is pressure ulcer risk.
- Check that the fastening straps are securely tied before use.
- Inspect the cushion and the skin regularly.

Accessories

- Standard polyester cover (*included*)
- Incontinence-protected cover

User weight	Max 150 kg (all sizes)
Material	Cold foam 55 kg/m ³ · Fire-retardant polyester cover
Design	Rounded rear edge · Two fastening straps · Adapted to kitchen chairs
Sizes	Seat width/depth per article number (DKHsbsd)
Weight	0.6–1.0 kg depending on size
Service life	Expected 5 years · Warranty 2 years

CE MD



Base:



Cover:



See page 2 for symbol explanations and full MDR info.

Technical documentation and MDR information

Supplementary regulatory information under Regulation (EU) 2017/745 on medical devices (MDR). Intended use, construction, user weight, accessories and care are stated on page 1.

Product identification and classification

Article number format: DKHsbdsdsh (where sb = seat width, sd = seat depth, sh = seat height in cm — e.g. DKH434075).

ISO code: 12 25 06 — Pressure redistributing cushion (ISO 9999).

Classification: Class I medical device under MDR (EU) 2017/745, rule 1.

Intended user: the user, following a needs assessment by a licensed prescriber.

UDI-DI: Each size has its own UDI-DI per GS1 (GTIN). Example: DKH434075 → GTIN 7333376000019.

Barcode: EAN-13.

Sterile/reuse: Non-sterile; reusable between users after reconditioning as instructed on page 1.

Testing and standards

Fire tested per: ISO 7176-16 (resistance to ignition of upholstered wheelchair parts).

Symbols: Labelling per EN ISO 15223-1 (medical devices) and ISO 3758 (textile care).

Quality management: Manufactured under ISO 13485.

Declaration of conformity: The manufacturer has issued an EU declaration of conformity — contact Nimacare AB for the current version.

Restrictions

- Users with an **existing category 3 or 4 pressure ulcer** on the seating surface — require assessment by a licensed prescriber before use.
- User weight exceeding the permitted limit for the size in question (see page 1).
- Use of a damaged cushion.

Storage, inspection and environment

- Inspect the cushion and cover regularly — a damaged product must not be used.
- Avoid use near open flames or strong heat sources.
- Store cool and dry. Avoid prolonged direct sunlight.
- Do not use solvents for cleaning.

Modification

The cushion must **not be cut down**. If another size is needed — choose another size from the range. Modification or cutting voids the CE marking and releases the manufacturer from liability.

Residual risks

As with all seat cushions, a risk of pressure injury remains with prolonged uninterrupted loading or if skin checks are omitted.

Incorrect size or orientation can increase the risk of pressure injuries, unstable sitting and malpositioning. Users with reduced sensation or circulation require particular follow-up.

Reporting of serious incidents

Serious incidents related to the use of this product must be reported to Nimacare AB (info@nimacare.se) and to the competent authority of the Member State where the user and/or patient is established. In Sweden, incidents are reported to the **Swedish Medical Products Agency (Läkemedelsverket)** (www.lakemedelsverket.se).

Manufacturer and regulatory contact

Nimacare AB

Sigurdvägen 4, 18254 Djursholm, Sverige

SRN (EUDAMED): SE-MF-000050989

Tel: +46 (0)70 635 50 70 · E-mail: info@nimacare.se · Web:

www.nimacare.se

Person responsible for regulatory compliance (PRRC) under MDR Art. 15 — contact info@nimacare.se.

Symbol explanations

Medical device symbols per **EN ISO 15223-1**. Wash and care symbols per **ISO 3758**.



Medical device



Consult instructions for use



Wipe clean — base only, no machine wash



Dry flat — cushion base dried flat



CE marking — complies with MDR (EU) 2017/745



Manufacturer



Machine wash 60 °C — cover only



Tumble dry allowed — cover, normal temperature

Document information: This product sheet constitutes both product information and instructions for use in paper form per MDR (EU) 2017/745, Annex I, section 23. Latest revision: 2026-07-30. Document ID: NIMA-DKH-EN-IFU-Rev2026-07-30. If this version differs from a later version published at www.nimacare.se, the later version applies. The product is UDI-DI labelled — check that the UDI on the label matches the delivery.